

Canfit TIPS and TRICKS



The “Action Tab”

www.vorum.com

Canada/USA Locations - (800) 461-4353

Other Global Locations - (604) 321-7277

The “Action Tab” will show you **all** of the modifications done, and the order in which they were performed. The list of modifications that appear in the “Action Tab” are referred to as the “Action Stack”.

Reviewing the Action Stack is very helpful when you want to see exactly what was done to each patient shape, when teaching staff members how you work in the *Canfit™* software, and for re-fabricating/updating a device that was made previously.


Read Below to see how the “Action Tab” can assist you.

Review modifications performed

Under the “Name” column the Action Tab shows the modifications performed and the “Details” column describes how the shape was modified.

For example: After applying a bend modification, you will see the name of the modification performed “Bend”, and a description of which portion of the shape was bent and by how much (in degrees).

The first modification applied appears at the bottom of the list in the Action Stack, the last modification is at the top.

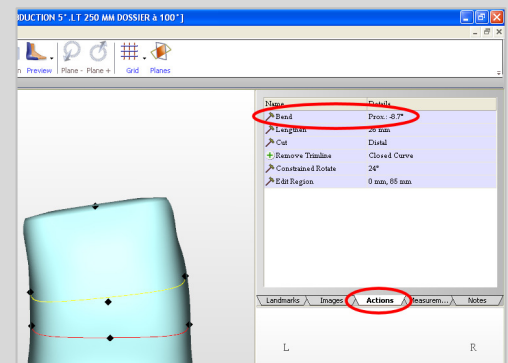


Undo modifications

1. Right click on the modification name in the Action Stack - it will become highlighted in red
2. Click on **undo** to remove the selected modification

If this modification is in the middle of the list, it will remove every modification that was performed after it.

After the modification is un-done it will become highlighted in pink and will remain in the action stack until a new modification is performed.

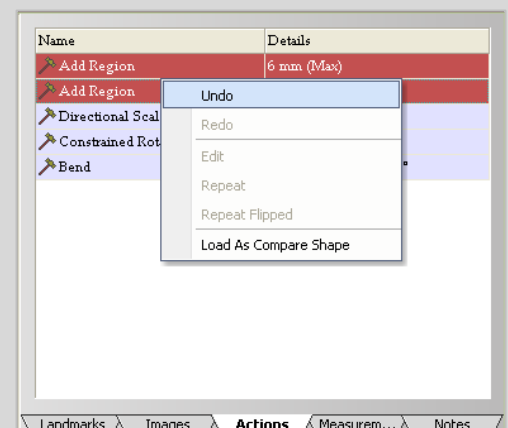

Redo modifications

1. Right click on the modification name in the Action Stack
2. Click **Redo** - The modification will appear on the shape exactly as it was before it was undone

You can redo any modifications that you undo, as long as they still appear in the Action Stack.

If you click on a modification in the middle of the list that has been undone, that modification and all of the others that appear below it in the list will be put back onto the shape.

Edit the last modification

1. Right click on the modification at the top of the Action Stack
2. Click **Edit**
3. Make any adjustments to the shape as needed

This will open up the last tool you were working with and display the modification parameters exactly as they were when you clicked the "OK" button.

Repeat the last modification

1. Right click on the modification at the top of the Action Stack
2. Click **Repeat**

The last modification will be performed with the same specifications as the previous. *Repeat* can be used for any modification, but is especially useful when performing the sanding action.

Repeat a flipped copy

1. Right click on the modification at the top of the Action Stack
2. Click **Repeat Flipped**

This can be used after a modification has been performed using the *Directional Scale* or *Constrained Rotate* tool. It is useful if your modification has resulted in a change on one side of the shape, and you want to repeat it with the same parameters on the other side of the shape.

Load your modified shape as your compare shape

1. Right click on any modification that appears in the Action Stack
2. Click **Load As Compare Shape**

This enables you to easily view the effects of a single modification. It is especially helpful when learning how to use the software, as it allows the user to see how each tool and modification affects the shape.

To return to the original compare shape

1. Click on the **Compare** tab at the top of the screen
2. Click **Default**

